4U Flow kapsel, spruta

Bruksanvisning (SE)

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4U Flow capsule, syringe

Instructions for use (ENG)

Product description: 4U Flow is a visible light cure easy flowing radiopaque nanohybrid composite for anterior and limited posterior restorations and for sealing. It is based on methacrylate-resin and inorganic filler particles of 0.05-1 µm. The total filler load is 62% and the total filler volume 47%. Radiopacity of LC FLOWFILL is 150 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine: ~ 100 % aluminium). The composite material meets the requirements of DIN EN ISO 4049, type 1, class 2, group 1.

Indications/ Intended use

- Restorations (class III, IV, V)
- Small class I and II restorations
- Lining of cavities
- Sealings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching. Ascertain the tooth shade while teeth are still moist and select the appropriate 4U

Flow shade. 4U Flow is shaded according to VITA®-Shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection, areas close to the pulp, should be covered with a thin layer of calciumhydroxide material. Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with etching gel and subsequent application of a light cure bonding agent is recommended. Otherwise use a self etching bonding agent before application of 4U Flow.

Placement of 4U Flow

Apply 4U Flow in the selected shade. Place it by using a suitable instrument. Matrix strips may be used.

Application from syringe:

Screw off the cap of the syringe and substitute it by one of the supplied NeedleTips. After use discard the NeedleTip and attach the cap to the syringe again. NeedleTips are for single use only due to hygienic reasons.

Curing of 4U Flow

For an optimum result apply 4U Flow in layers of max. 1 mm. Light cure each increment separately. By using a polymerization unit with a light intensity of at least 1000mW/cm² and a wavelength range of 400 to 500 nm (blue light) cure each increment as follows:

Lighter shades (e.g. A1, A2, B2) 20 seconds

Darker shades (e.g. A3.5) 30 seconds

Hold the light emission window as close as possible to the filling material.

Finishing

Contour with fine diamonds, stones or burs. Polish to high gloss with discs or rubber points. Interproximal finishing is accomplished by fine grit finishing strips.

Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Additional Notes/warnings

- Application caps are for single use only due to hygienic reasons.
- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of the composite restorative.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

Composition

Dimethacrylates, dental glass, silicon dioxide, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

S&C Polymer Silicon- und Composite Spezialitäten GmbH warrants this product will be free from defects in material and manufacture. S&C Polymer Silicon- und Composite Spezialitäten GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and S&C Polymer Silicon- und Composite Spezialitäten GmbH's sole obligation shall be repair or replacement of the S&C Polymer Silicon- und Composite Spezialitäten GmbH product.

Limitation of Liability

Except where prohibited by law, S&C Polymer Silicon- und Composite Spezialitäten GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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