en Instruction for use

Patient target group

Children, adolescents, adults and older patients

Intended users

Dentist

Composition

Qualitative information according to the information in the biocompatibility:

Calcium hydroxide, water, barium sulphate, hydroxyethyl cellulose

Quantitative data: Substances

(Quantity according to biocompatibility):

45 % calcium hydroxide,

5 % barium sulphate

Intended purpose of the product

Simplee Calciumhydroxide Paste is an aqueous calcium hydroxide paste.

Indications

Treatment of caries-free dentin close to the pulp, deep caries treatment, indirect pulp capping, direct pulp capping, pulpotomy and temporary insertion in the root canal.

Contraindications

The use of Simplee Calciumhydroxide Paste is contraindicated: in case of pulp opening/exposure in carious dentin, in case of inflammatory pulp changes, in case of hypersensitivity to one of the ingredients.

Performance characteristics of the product

Alkalisiation through calcium hydroxide

Specific precautions

None

Application

The amount of Simplee Calciumhydroxide Paste to be administered depends on the respective application:


1. Deep caries treatment / indirect pulp capping - Simplee Calciumhydroxide Paste is applied to the cavity floor as a thin film without pressure. A temporary or final filling is then place
2. Direct pulp capping - After the working area has been dried, Simplee Calciumhydroxide Paste is applied and a suitable covering filling is then place to seal the tooth permanently.
3. Vital amputation - Simplee Calciumhydroxide Paste creates a favourable condition for healing of the remaining pulpal stumps due to the recognised localised corrosive effect of calcium hydroxide on the residual pulp.

Danger

H318 – causes serious eye damage

H335– may cause respiratory irritation

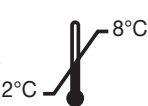0482

Made in Germany

4. Underfilling / pulp protection - The space-saving radiopaque Simplee Calciumhydroxide Paste also forms a thin whitish insulating layer against cement acids or free monomer in the anterior region under silicate or resin fillings as pulp protection
5. Dentin wound dressing for ground vital teeth - Simplee Calciumhydroxide Paste is applied to the prepared tooth stump with a brush or cotton swab and carefully dried with warm air. Avoid contact with the gingiva. The brush should first be dipped in boiled water or distilled water to dilute the paste slightly.
6. Medical insertion into infected root canals. Each root canal should be filled with Simplee Calciumhydroxide Paste without air bubbles. If necessary, the insert must be renewed after 6 to 8 weeks.

Warnings

Residual risks

Simplee Calciumhydroxide Paste must always be kept tightly closed. Do not apply increased pressure to the opened pulp during application. Calcium hydroxide causes local irritation and corrosive effects on skin, eyes and mucous membrane. Calcium hydroxide particles on the cornea and in contact with the eye must be removed immediately by suitable mechanical means, then rinse immediately and thoroughly with water. This medical device must not be used after the expiry date. This medical device must be kept out of the reach of children.

Side effects

Adverse side effects are not known. Risks in case of overdose are not known.

Interactions

Interaction of the product with other medical devices, medicinal products or other substances is not known.

Storage

Important notice:

Upon receipt, the product must be immediately stored under refrigerated conditions at +2°C to +8°C.

Minimum shelf life

Do not use after the expiry date.

Variants

REF 156 117 - 1 x syringe with 2.0 ml paste + cannulas

Disposal

Unused material can be disposed of with household waste.

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

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