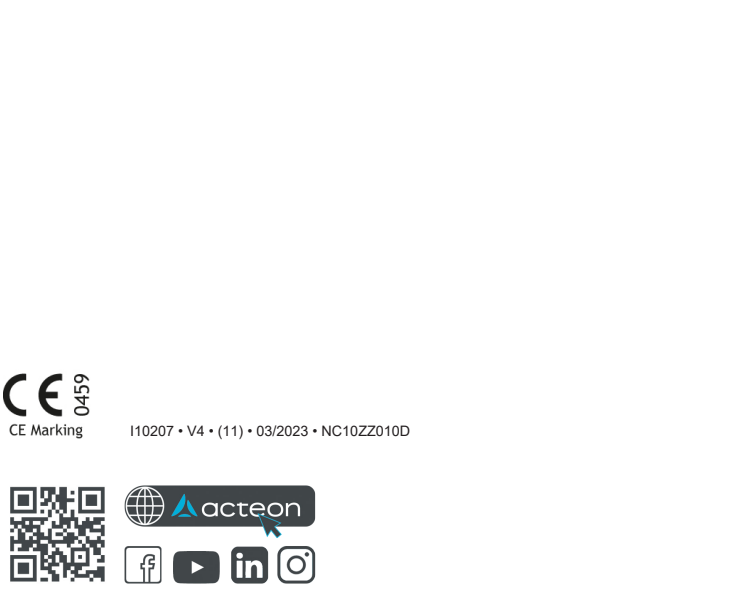
SATELEC ■ A company of **ACTEON** Group
17, avenue Gustave Eiffel ■ ZI du Phare ■ 33700 MERIGNAC ■ FRANCE
Tel. +33 (0) 556 340 607 ■ Fax. +33 (0) 556 349 292 ■ info@acteongroup.com

ENGLISH

Caution: The United States Federal Law restricts the use of this device to qualified dental health professionals only or professionals capable and qualified or under their control.

1 - PRESENTATION
AIR-N-GO® «PERIO» powder: Powder used during the sub-gingival treatment of periodontitis and peri-implantitis.
AIR-N-GO® «PERIO» powder was designed to be used exclusively:
• During the sub-gingival treatment of periodontitis and peri-implantitis, as a follow up to an initial therapy, once inflammation and/or bleeding have subsided.
• During the prevention and control of periodontitis and peri-implantitis
This powder is used exclusively with the AIR-N-GO® device, either in the AIR-N-GO® «PERIO» sub-gingival configuration for the treatment of periodontitis and peri-implantitis, or, in the supra-gingival configuration for the prevention and control of periodontitis and peri-implantitis. Please read and adhere to the user instructions for the device.
AIR-N-GO® «PERIO» powder packaging: 160 g multi-dose bottle, with spout.

2 – GUIDELINES
• Removal of sub-gingival plaque from tooth roots or implants. AIR-N-GO® «PERIO» powder cannot be used to remove built-up tarter.
• Treatment of periodontitis and peri-implantitis using the AIR-N-GO® device in sub-gingival configuration (PERIO).
On a natural tooth, AIR-N-GO® «PERIO» powder combined with the use of the PERIO nozzle is recommended exclusively as a follow-up to an initial therapy, on pockets deeper than 3mm and less than 10mm in depth.
We strongly recommend that an x-ray is taken to measure and check that the height of the alveolar bone at the level of the tooth apex is at least 3mm.
Do not use AIR-N-GO® «PERIO» powder and the PERIO nozzle to treat cases of radicular periodontitis. The first stage of the initial therapy involves probing, cleaning and debriding periodontal pockets using ultrasonic inserts and/or manual instruments.
AIR-N-GO® «PERIO» powder is also used, generally seven to ten days after the first appointment, when inflammation and/or bleeding have subsided.
It is important to make sure that the collagen fibres have not started to reattach themselves to the root and that tissues show as little inflammation as possible.
This also applies to the treatment of a peri-implantitis, i.e. the AIR-N-GO® «PERIO» powder combined with the PERIO nozzle is only used after the probing, cleaning and debriding of the implant pocket using ultrasonic inserts and/or manual instruments.
Adhere to the same precautions for the treatment of implants as those specified above for teeth.
In the case of periodontitis or peri-implantitis:
AIR-N-GO® «PERIO» treatment is carried out once the dentist has checked that there is no longer any bleeding with probing and that the pocket open-bite allows for nozzle insertion.
SATELEC® ultrasonic inserts TK1-1S (P/N F01001), PerioPrécision P2L (P/N F00090)/P2R (P/N F00091) for natural teeth and/or carbon inserts PH1 (P/N F00702)/PH2L (P/N F00705) and PH2R (P/N F00706) for implants may be used for pocket debridement.
• Gentle cleaning of teeth or implants during the prevention or control of periodontitis and peri-implantitis, using the AIR-N-GO® device in its supragingival configuration.

3 - PROPERTIES
AIR-N-GO® «PERIO» powder is made up of extremely fine grains of an average size of 25 µm. It does not cause pain or excessive bleeding and is perfectly adapted to sub-gingival treatments. It creates a pleasant feeling in the mouth.

4 – COMPOSITION: Glycine, bases

5 - PRECAUTIONS FOR USE
Before using the powder, ask your patient questions about their general health.
• Comply with the conditions of use recommended in the AIR-N-GO® device instructions • Patient and dentist must wear suitable protection such as goggles and masks • Do not point the spray directly towards the gum • The patient must rinse his/her mouth with an antiseptic solution before and after the treatment • Apply Vaseline® to the patient's lips before polishing • Use a large suction cannula to be held near the area to be treated (reduction of bacterial air contamination) • In the event of accidental contact with the skin or eyes, rinse with plenty of water • Do not swallow.

6 – CONTRAINDICATIONS
• Known allergy to one of the ingredients • Patients suffering from endocarditis • Patients with an immune deficiency • Patients taking antibiotics or undergoing chemotherapy or radiotherapy treatment, etc. • Diabetic, asthmatic or suffering from respiratory problems • Haemophiliacs • Pregnant or breastfeeding women • Radicular periodontitis.

7 – USER INSTRUCTIONS
Use:
1 - With the AIR-N-GO® «PERIO» device in subgingival configuration.
Caution: For the treatment of periodontitis and peri-implantitis, adhere to the recommendations for use of the AIR-N-GO® device in its subgingival configuration (PERIO).
• Use the surgical aspiration cannula and maintain it near the treated area for cleaning the operatory area.
• Treat your patient as required.
• Each tooth or implant must be treated individually.
• Make sure there is no excessive bleeding during probing.
• Insert the nozzle to the bottom of the pocket exerting the same amount of pressure as with a periodontal probe.
• Activate AIR-N-GO® by pressing the control pedal on your dental chair, only when the PERIO nozzle is in the pocket.
• Gently move the nozzle upwards moving it slowly from side to side without pressure for 5 seconds per side/ surface.
• Leave the AIR-N-GO® device on even when you have gone past the top of the pocket. The gingival festoon must be gently treated in order to obtain optimum results.
• Repeat the operation on the surfaces of the tooth or implant to be treated.
Caution: The air/powder spray continues for a few seconds after releasing the control pedal. Take this into account and wait before removing the nozzle from the patient's mouth in order not to disseminate bacteria in the room.
• During the healing period, we recommend you prescribe the patient a digluconate solution containing 0.2% chlorhexidine, to be taken twice daily for 3-4 weeks.

2 - With the AIR-N-GO® device in supra-gingival configuration.
For the prevention or control of periodontitis and peri-implantitis, adhere to the recommendations in the AIR-N-GO® device user instructions concerning the AIR-N-GO® device in supra-gingival configuration.
• Put the AIR-N-GO® «PERIO» powder in the Blue tank on the AIR-N-GO® device, in supra-gingival configuration. Assemble and use the SUPRA 120° or 90° nozzle.
• Gently clean the tooth or implant in compliance with the recommendations in the user instructions concerning the device in supra-gingival configuration.

Servicing:
After each use of AIR-N-GO® «PERIO» powder,
- Press the «STOP POWDER» button on the tank to clean the device tubes.
- Immerse the used nozzle (PERIO or SUPRA) in an ultrasonic container filled with an undiluted Listerine type disinfectant (or other ethanol-based mouthwashes) for 10 minutes to dissolve the glycine and prevent risks of clogging.

8 - PRESERVATION
Do not exceed the use-by date marked on the exterior packaging. Close the bottle securely after use. Store at ambient temperature (15-25°C/59-77°F, 5-75 %), away from humidity and light.

9 – DISPOSAL OF UNUSED POWDER
Do not dispose of down drains or in rivers. Do not dispose of with household waste. Empty all powder from the tank into the chair spittoon or the sterilisation room sink.

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